epiAge

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EN Table of Contents

Advisory.....	04
Instructions for Use.....	05
Product Information.....	10

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ADVISORY

- ▶ Please wait at least 3 months before taking the test if you have had any serious illnesses, such as COVID-19, or have taken the COVID-19 vaccine. Otherwise, the result of the test may be severely distorted. Also, bacterial infections, such as bronchitis, and other viral infections like the flu and the common cold can also seriously impact the quality of the saliva sample.
- ▶ **Do not** swallow the buffer solution and avoid contact with eyes.
- ▶ Keep out of the reach of children.

INSTRUCTIONS FOR USE

BEFORE THE TEST

REGISTER THIS TEST KIT AT www.epi-age.de/register
AND SELECT YOUR DESIRED LANGUAGE.

OTHERWISE, YOUR TEST CANNOT BE PROCESSED! YOU
WILL FIND THE BARCODE ON THE TEST TUBE AS WELL AS
ON THE BACK COVER OF THIS BROCHURE.

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ON THE DAY OF THE SALIVA SAMPLE COLLECTION:

- ▶ AVOID COLLAGEN-BASED SUPPLEMENTS!

30 MINUTES BEFORE COLLECTION:

- ▶ DO NOT EAT OR DRINK ANYTHING,
- ▶ DO NOT BRUSH YOUR TEETH, SMOKE OR CHEW GUM,
- ▶ RINSE MOUTH WITH WATER.

SAMPLE COLLECTION

STEP 1

Gently rub your cheeks behind the molars to stimulate saliva secretion.

STEP 2

Remove the collection tube with the blue funnel from the box. Collect 2 ml of saliva (bubbles **do not** count!) by spitting into the tube. After collection, make sure the tube remains upright and lean it securely.

STEP 3

Get the other tube with the buffer solution out of the box and unscrew the cap. Slowly pour the buffer solution from the tube into the funnel of the collection vessel.

STEP 4

Next, unscrew the blue funnel from the collection tube (which now contains saliva and buffer solution). Then take the cap provided and screw it tightly onto the tube.

Finally, shake the tube at least 10 times to blend the solution thoroughly. This ensures saliva preservation.

STEP 5

Place the collection tube in the enclosed plastic bag with the biohazard symbol . Seal the bag.

STEP 6

Then place the bag containing the saliva sample into the self-addressed envelope provided. The sample is now ready for dispatching. No stamp required for shipping from Germany.

STEP 7

Dispose of the funnel and empty tube according to local regulations.

WHEN SHIPPING FROM OUTSIDE OF GERMANY, PLEASE AFFIX ADEQUATE STAMP/POSTAGE ONTO THE ENVELOPE.

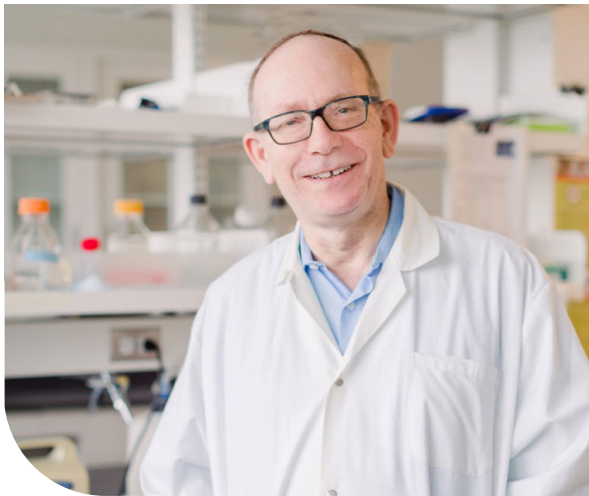
PLEASE NOTE THAT THE BUBBLE ENVELOPE HAS THE STATUS OF A LETTER AND, AS A MEDICAL SAMPLE, IT DOES NOT REQUIRE A CUSTOMS DECLARATION. SO PLEASE DO NOT SEND IT AS A PARCEL/ PACKAGE WITH A CUSTOMS DECLARATION OR BY REGISTERED MAIL!

ANY COSTS INCURRED AS A RESULT WILL BE AT YOUR OWN EXPENSE.

IF YOU WISH TO USE A COURIER SERVICE SUCH AS e.g. UPS, FedEx, ETC., PLEASE USE OUR COMPANY HQ ADDRESS:
epiAge Deutschland UG
Jakob-Klar-Str. 4, 80796 Munich, Germany.

Should you have any questions about your sample or the result, please contact us via email and state your secure ID number.

E-Mail: support@epi-age.de
www.epi-age.de/support

Professor Moshe Szyf
McGill University

HOW OLD AM I REALLY? epiAge has the answer.

Discover your real age – The epiAge test reveals your biological age. The patented epiAge test procedure uses methylation markers on your DNA to determine your actual epigenetic age – with the utmost scientific accuracy.

YOUR AGE IS MORE THAN A NUMBER.

We don't just age chronologically – Biological age is an important pointer on your way to a long and healthy life. Many factors, such as daily habits or stress levels, affect it. Once your epigenetic age is determined, you can use it to make meaningful lifestyle changes. Indeed, the methylation markers on your DNA are reversible. So this is the good news: you have the power to influence your age!

age example: chronological age = 45
 biological age = 37

TURN BACK YOUR BIOLOGICAL CLOCK.

Impetus for change – Any doctor will confirm that certain behaviours have positive effects on your health - and thus also on your ageing process: eat a balanced, healthy and varied diet, get enough regular exercise, make sure you get plenty of regenerative sleep, avoid stress and refrain from smoking as well as excessive alcohol consumption. You can then perceive your epiAge test result as a confirmation of your efforts or a catalyser for change.

YOUR AGE PUT TO THE TEST.

Trust is good. epiAge testing is better – The epiAge test provides you with a personalised age check. Depending on whether your biological age is higher or lower than your chronological age, you can evaluate your own lifestyle and determine whether changes are needed. Even better: with a follow-up test, you can monitor your efforts by checking their effects on your ageing process. So why not slow it down by a couple of years?

THIS IS HOW THE epiAge TEST WORKS.

Your **scientifically precise age** – epiAge offers you a user-friendly DNA analysis kit to calculate your biological age. Based on a simple saliva test, you will obtain an estimation of your epigenetic age within just a few weeks. epiAge Germany collaborates with the lab of **Professor Moshe Szyf** – one of the leading epigeneticists worldwide. So, take advantage of this cutting-edge scientific expertise to plan new health-promoting measures and anti-ageing strategies as well as to regularly monitor their effectiveness.

HKG *epi* THERAPEUTICS Ltd

ENROL EPIGENETICS FOR YOUR SUCCESS.

About epiAge - The epiAge test was developed by Professor Moshe Szyf from McGill University in Montreal. His HKG epiTherapeutics laboratory analyses your sample according to state-of-the-art research findings. On the basis of sound scientific expertise, epiAge Germany provides you with a precise epigenetic tool to determine your personal biological age. Stay young!

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HKG epi THERAPEUTICS Ltd
Shenzhen MandeLab Co., Ltd.
Manufactured in China for HKG epiTHERAPEUTICS Ltd.